



Battery replacement instructions for your Bell Smart Home equipment.

Enclosed you will find the replacement battery for your AlarmCare Fall Sensor and the step-by-step instructions on how to replace it.

Battery Disposal

We encourage you to dispose of your old battery in an environmentally friendly way. For more information about the designated battery types and recycling depot locations, please visit www.call2recycle.ca.

Instructions

Please start by disarming your sensor.

AlarmCare Fall Sensor

Battery Type: 1 x 3V CR-2477

1. Use a Phillips screwdriver to remove the screw located on the back of the unit.
2. Remove the battery cover.
3. Observe the battery polarity.
4. Install the new battery.
5. Replace the cover and tighten the screw.
6. Contact Technical Support to confirm the battery replacement was successful and the sensor is in proper working order. Contact numbers are located at the end of this document.

For any additional questions about your Bell Smart Home equipment, visit bell.ca/smart-home.
Or call our Technical Support team using the following numbers:
Ontario and Québec: 1 877 267-2004. Manitoba: 1 866 949-0078.
New Brunswick, Nova Scotia, Newfoundland and Labrador and PEI: 1 844 474-2023.

